

Tubifast® 2-Way Stretch®


A high quality tubular bandage for patients complete freedom of movement

- Contours to any shaped limb or torso thanks to 2-Way Stretch® Technology
- Light elasticity in both vertical and horizontal directions allowing patients of complete freedom of movement¹
- Self-retaining tubular construction: no pins or tapes are needed
- Easy to use: can be cut to required length
- Latex free
- Washable and re-usable
- Available in 5 colour-coded widths to fit any part of the body




Tubifast® 2-Way Stretch® tubular bandages are designed for dressing retention without the need for pins or tapes, patch wrapping (wet or dry) and as an undercast stockinette. Tubifast can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa².


How to use Tubifast® 2-Way Stretch®

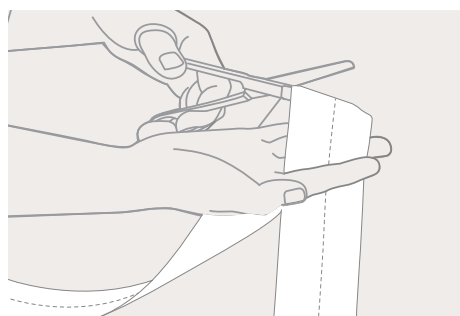

1. Choose the correct width bandage, using the quick reference colour coding, and cut to length.

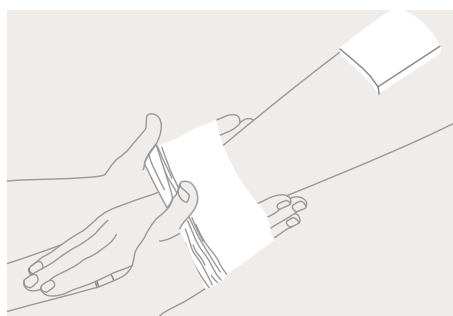

2. Stretch Tubifast® 2-Way Stretch® over the affected area.

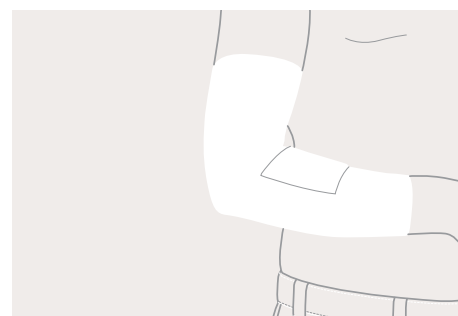

3. Position Tubifast 2-Way Stretch to cover the dressing. Illustrated, step-by-step instructions are available for more advanced applications.

How Tubifast 2-Way Stretch works

Tubifast® 2-Way Stretch® is a self-retaining tubular construction that allows air to circulate over the skin. Tubifast 2-Way Stretch provides a light elasticity in both vertical and horizontal directions allowing patients of complete freedom of movement.

Benefits of Tubifast 2-Way Stretch

- Contours to any shaped limb or torso thanks to 2-Way Stretch® Technology
- Allows complete freedom of movement
- Easy to use: can be cut to required length
- Self-retaining with no pins or tapes necessary
- Latex free, non-sensitising and non-irritating
- Reusable after washing
- Available in 5 colour-coded widths to fit any part of the body

Areas of use

Tubifast 2-Way Stretch tubular bandages are designed for dressing retention without the need for pins or tapes; it is particularly suitable for holding dressings in place on difficult areas such as stumps of amputee patients.

The product is also designed for patch wrapping (wet or dry) and as an undercast stockinette. Tubifast 2-Way Stretch can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa²

Fibre content

Viscose, Elastane and Polyamide

Tubifast 2-Way Stretch Ordering Information

Product	Width x Length	Limb ↔	Art no.	PIP Code	NHS Code
RED LINE	3.5cm x 1m	9-18cm	2480	005-0088	-
Small limbs	3.5cm x 10m	9-18cm	2434	026-8698	EGP115
GREEN LINE	5cm x 1m	14-24cm	2481	005-0039	-
Small and medium limbs	5cm x 3m	14-24cm	2485	255-1885	-
	5cm x 5m	14-24cm	3555	255-5803	-
	5cm x 10m	14-24cm	2436	026-8680	EGP117
BLUE LINE	7.5cm x 1m	24-40cm	2482	004-9924	-
Large limbs	7.5cm x 3m	24-40cm	2486	255-1877	-
	7.5cm x 5m	24-40cm	3556	255-5811	-
	7.5cm x 10m	24-40cm	2438	026-6924	EGP119
YELLOW LINE	10.75cm x 1m	35-64cm	2483	023-6539	-
Extra-large limbs, heads, children's trunks	10.75cm x 3m	35-64cm	2487	255-1869	-
	10.75cm x 5m	35-64cm	3557	255-5829	-
	10.75cm x 10m	35-64cm	2440	000-6221	EGP121
PURPLE LINE	20cm x 1m	64-130cm	2479	324-1452	-
Adult trunks	20cm x 5m	64-130cm	3558	324-1486	-
	20cm x 10m	64-130cm	2444	-	EGP107

Tubifast 2-Way Stretch is packaged in single packs

Washing instructions



References: 1. Eytier C, Gazeau E, Beneteau G, Verfaillie G. Convenience and tolerance of the combination of a soft silicone foam dressing and a two-way stretch tubular bandage in the management of local wounds. *Journal des plaies et cicatrisations* 2013;18(88):38-44. 2. Mölnlycke Health Care. Data on file.

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00. The Mölnlycke, Tubifast and TwoWay Stretch trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2020 Mölnlycke Health Care AB. All rights reserved. HQIM002144