

Mepitac[®]

Fixation tape with soft silicone micro-adherence

Knitted fabric


- Conformable
- Flexible
- Easy to cut to size

Breathable polyurethane film

- Waterproof

Safetac[®] soft silicon micro-adherent technology

- Does not cause skin damage or irritation
- Adheres gently to the skin
- Minimizes trauma and pain on removal
- Leaves no residue

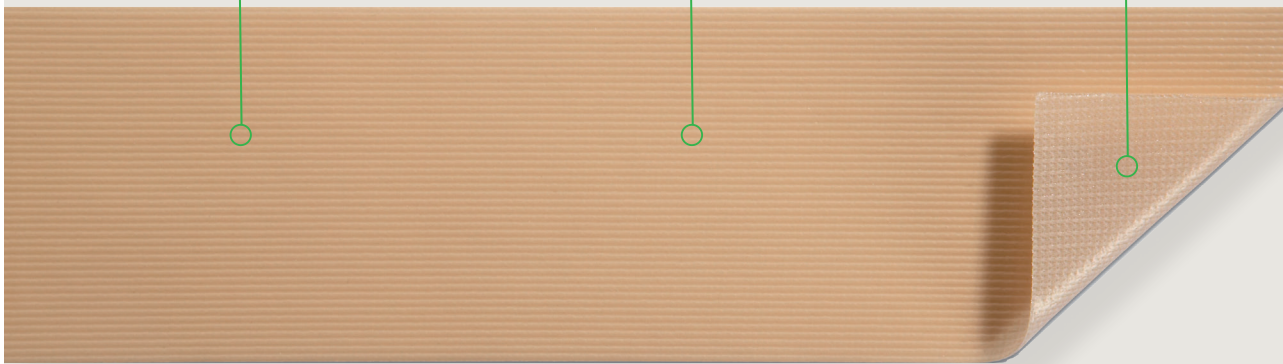

Safetac[®]
TECHNOLOGY

- Minimizes trauma and pain on removal
- Ideal fixation for patients with fragile and sensitive skin
- Conforms well to body contours
- Can remain in place for several days which allows cost-effective and undisturbed

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex[®], Mepitel[®], Mepiform[®] and Mepitac[®]. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.


* A unique proprietary technology exclusive to Mölnlycke Health Care

Safetac[®]
TECHNOLOGY


Mepitac[®]


Mölnlycke[®]


Directions for use

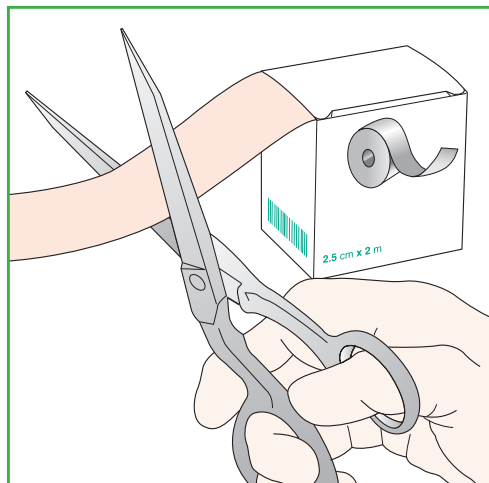

1. Ensure skin is clean and dry and free from topical preparations. Cut to the desired length.

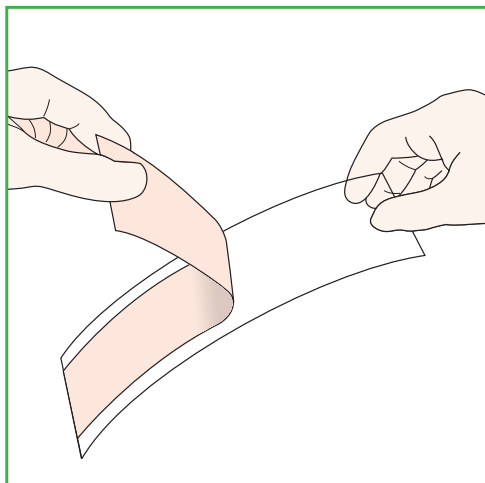

2. Remove the protective release film.

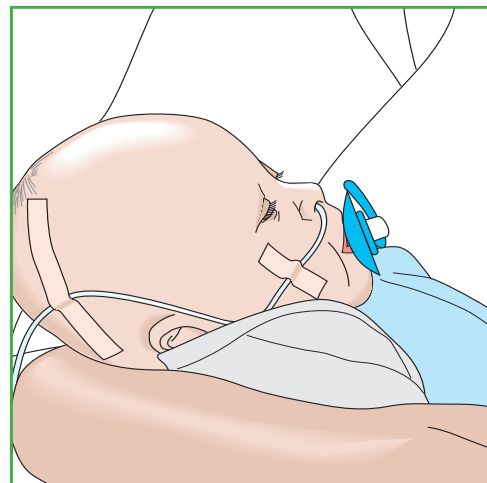

3. Apply Mepitac® without stretching. If inspection or repositioning is necessary, Mepitac® can be lifted and re-applied.

How Mepitac® works

Mepitac® is highly conformable and easy to apply and remove. The Safetac® layer assures gentle yet secure fixation and minimizes skin trauma and pain during removal. It can be lifted and re-applied if necessary. Mepitac® leaves no residue and is non-irritating.

Mepitac® provides ideal fixation for patients with fragile and sensitive skin, allergies and skin conditions such as eczema or dermatitis. It is suitable for individuals requiring repeated application and removal of tape over the same area (i.e. neonates, wound, ostomy and dialysis patients).

Benefits of Mepitac®

- Atraumatic to the skin on removal
- Conforms well to body contours
- Leaves no residues
- Waterproof and breathable
- Can remain in place for several days and can be repositioned which promotes cost-effective wound healing

Mepitac® Assortment

Art. no	Size	Pcs/Box	Pcs/Case
298300	2cm x 3m	1	12
298400	4cm x 1.5m	1	12

Indications for use

Mepitac® is designed for fixation of drains, tubes, probes, electrodes, IV cannulae and dressings. Additionally, Mepitac® can be used under devices such as tubes or cannulae to protect skin.

Precautions

Mepitac® is not intended for fixation of life supporting devices or for use on arterial catheters or cannulae.

Note:

The adhesion of Mepitac® may decrease when used in humid conditions such as incubators or for diaphoretic patients.

