


Mepitel® Film

Skin protective film dressing
With Safetac® technology

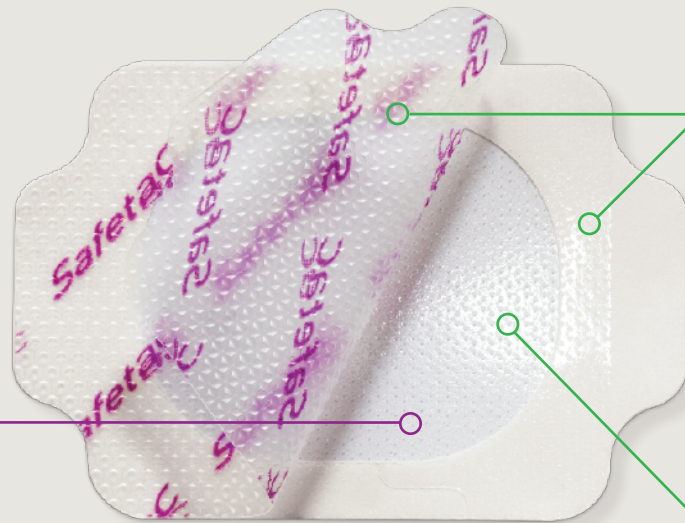

Safetac® layer

- Reduces pain and trauma before, during and after dressing change^{1,2,3,5,6}
- Does not adhere to the moist wound bed but to dry skin only²
- Seals the wound margins and reduces risk of maceration^{3,4}

Safetac
TECHNOLOGY

Supporting **protection backing** and **paper frame** for ease of application

- Transparent, breathable, elastic polyurethane film for patient comfort and optimized wound healing
- Shower proof, bacteria and viral barrier (microbes →25nm)

- Mepitel® Film is a gentle, transparent breathable film dressing for skin protection.
- Mepitel® Film minimizes pain and trauma at dressing changes.^{1,2,3,4,5,6}
- Mepitel® Film's application system is designed for ease-of-use.

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.

* A unique proprietary technology exclusive to Mölnlycke Health Care

Safetac
TECHNOLOGY

Mepitel® Film


Mölnlycke[®]

Directions for use



Gently cleanse the wound; dry surrounding skin. Remove the protection foil (printed Safetac®) to expose the adhesive.

For best results, Mepitel® Film must overlap the dry surrounding skin by 1-2 cm for the sizes up to 10 x 12 cm. For the larger sizes, dressing must overlap onto the surrounding skin by 5 cm to fixate securely.



Position the dressing. Mepitel® Film can be repositioned as long as the paper frame is intact. Remove the white paper frame. Do not stretch the dressing when applying.



Firmly smooth the dressing onto the skin. Mepitel® Film can be removed without causing skin stripping and pain.

How Mepitel® Film works

Mepitel® Film provides a flexible, transparent covering to protect the skin or wound from microbial contamination, fluid strike-through and from other external contamination while conforming to surface irregularities and body contours. Mepitel® Film maintains a moist environment even though the vapour permeability allows excess wound moisture to pass away from the skin. The Safetac® layer provides instant adhesion that does not require extra pressure to fixate well in place. The Safetac® layer also ensures atraumatic dressing changes. Mepitel® Film leaves no residue and the adhesion level does not increase over wear time.

Indications*

Mepitel® Film is designed for the management for a wide range of superficial wounds such as pressure injuries stage/grade I and II, superficial burns, superficial skin injuries and it may help to prevent skin breakdown. Mepitel Film protects fragile and sensitive skin.

Mepitel® Film can also be used as a protective cover for open surgical wounds (e.g. abdomen), fixate primary dressing as a secondary dressing and be used in combination with gels and ointments.

Frequency of change

Mepitel® Film may be left in place for several days depending on the condition of the skin or wound, or as indicated by accepted clinical practice.

Mepitel® Film Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
296170	6 x 7	10	50
296270	10 x 12	10	70
296470	10 x 25	10	50
296670	15 x 20	10	70

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Mölnlycke
1-800-494-5134
www.molnlycke.ca

Mölnlycke Health Care Canada
2010 Winston Park Drive, Suite 100 Oakville, ON L6H 5R7. Phone : 905-829-1502
© 2023 Mölnlycke Health Care AB. All rights reserved.

Benefits of Mepitel® Film

- Minimizes pain and trauma at dressing changes
- Limits the risk of damaging newly formed tissue
- Transparent for easy inspection
- Conforms well to body contours, promoting patient comfort during wear
- Bacteria and viral barrier where microbes larger than 25nm
- Low potential for skin irritation and allergy
- Leaves no residue
- Fluid impermeable
- Shower proof

Precautions*

- Do not use as primary fixation for IV, cannulae, ports or other infusion and or life sustaining devices.
- Mepitel® Film is not a wound contact layer product and does not allow wound exudate to-pass through the dressing to a secondary layer.
- If more than one product is used and they are overlapped, the vapour permeability goes down; this can lead to excess moisture not being allowed to pass away from the skin.
- In case of signs of clinical infection, consult a health care professional for adequate infection treatment.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.




Mölnlycke®

CEWC01492301