

How to apply on the hand

For this application, you will need one Mepilex® Ag 10x20cm dressing, one Mepilex® Ag 15x15cm dressing, and one Mepilex® Ag 10x10cm dressing.



1. Use Mepilex Ag 10x20cm to cover the hand.



2. Fold the dressing and make four small, equidistant cuts (about 1.5cm long).




3. Apply the dressing by making each finger slide into the newly created holes.



4. Use Mepilex Ag 15x15cm to cover the fingers.



5. Cut the dressing in four slats of the same width.

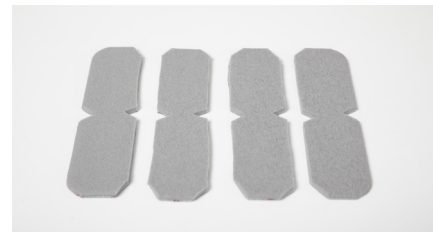

6. Cut the corners of each slat at an angle and cut out two small, aligned triangles (about 0,5cm deep) on each side of the dressing.



7. Apply one part of the dressing to the finger and fold it back using the newly created small triangles.



8. Use Mepilex Ag 10x10cm to cover the thumb.



9. Cut the dressing on both sides.



10. Cut another of the remaining round corners to have three straight sides.



11. Roll the dressing around the thumb to cover it.



Assortment overview

Mepilex® Ag

An antimicrobial foam dressing for low-to-medium exuding burns. It absorbs exudate and maintains a moist wound environment. It provides fast (from 30 minutes), sustained and broad-range antimicrobial action for up to 7 days, as shown in vitro.

Size cm	Size inch	Pcs/shelf	cont Pcs/transp. cont.	Art. no.
10 x 10	4 x 4	5	70	287110
10 x 20	4 x 8	5	45	287210
15 x 15	6 x 6	5	35	287310
20 x 20	8 x 8	5	25	287410
20 x 50	8 x 20	2	10	287500



Note: Not all articles are available in every country. Please contact your local Mölnlycke Health Care representative for information about articles available in your country.

Tubifast®

A 2-way stretch tubular bandage designed for dressing retention and skin covering for any part of the body. It provides light elasticity both length and widthways, allowing complete freedom of movement.

Product	Width x Length	Limb	Art. no.
RED LINE – Small limbs	3.5cm x 10m	9-18cm	2434
GREEN LINE – Small and medium limbs	5cm x 10m	14-24cm	2436
BLUE LINE – Large limbs	7.5cm x 10m	24-40cm	2438
YELLOW LINE – Extra-large limbs, heads, children's trunks	10.75cm x 10m	35-64cm	2440
PURPLE LINE – Adult trunks	20cm x 10m	64-130cm	2444



Tubifast 2-Way Stretch is packaged in single packs.

Tubifast® garments

Product	Age	Art. no.
Vests with integrated mittens	6-24 months	992007
	2-5 years	992008
	5-8 years	992009
	8-11 years	992010
Tights	11-14 years	992011
	6-24 months	992012
	2-5 years	992013
Leggings	5-8 years	992014
	8-11 years	992015
	11-14 years	992016
Socks	1 size: 2-14 years	992017
Gloves	Child XS	5922
	Child Small	5923
	Child Medium-Large/Adult Small	5921
	Adult Medium-Large	5920



References:


1. Gee Kee EL, Kimble RM, Cuttle L, Khan A, Stockton KA. Randomized controlled trial of three burns dressings for partial thickness burns in children. Burns 2015, 41(S1):946-955. 2. Silverstein P, Heimbach D, Meites H et al. An open, parallel, randomized, comparative, multicenter study to evaluate the cost-effectiveness, performance, tolerance, and safety of a silver containing soft silicone foam dressing (intervention) vs silver sulfadiazine cream. J Burn Care Res 2011, 32(6): 617-626. 3. Gee Kee EL, Stockton K, Kimble RM et al. Cost-effectiveness of silver dressings for paediatric partial thickness burns: An economic evaluation from a randomized controlled trial. Burns 2017, 43(4): 724-732. 4. Jaggawala S, Harish V, Roberts S et al. Treatment of partial thickness burns: a prospective, randomised controlled trial comparing Biobrane, Acticoat, Mepilex Ag and Aquacel Ag. J Burn Care Res 2020, 42(5): 934-43. 5. Tang H, Lv G, Fu J et al. An open, parallel, randomized, comparative, multicenter investigation evaluating the efficacy and tolerability of Mepilex Ag versus silver sulfadiazine in the treatment of deep partial-thickness burn injuries. J Trauma Acute Care Surg 2015, 78(S1): 1000-1007. 6. Nischwitz SP, Luze H, Popp D, Winter R, Draschl A, Schellnegger M, Kargl L, Rappi T, Giretzlehner M, Kamolz LP. Global burn care and the ideal burn dressing reloaded—A survey of global experts. Burns 2021, 47: 1665-1674.

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, Gamlestadsvägen 3C, SE-402 52 Göteborg, Sweden. Phone +46 31 722 30 00. The Mölnlycke, Mepilex, Mefix and Tubifast trademarks, names and logotypes are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2023 Mölnlycke Health Care AB. All rights reserved. H01M004174



The Power of Gentle

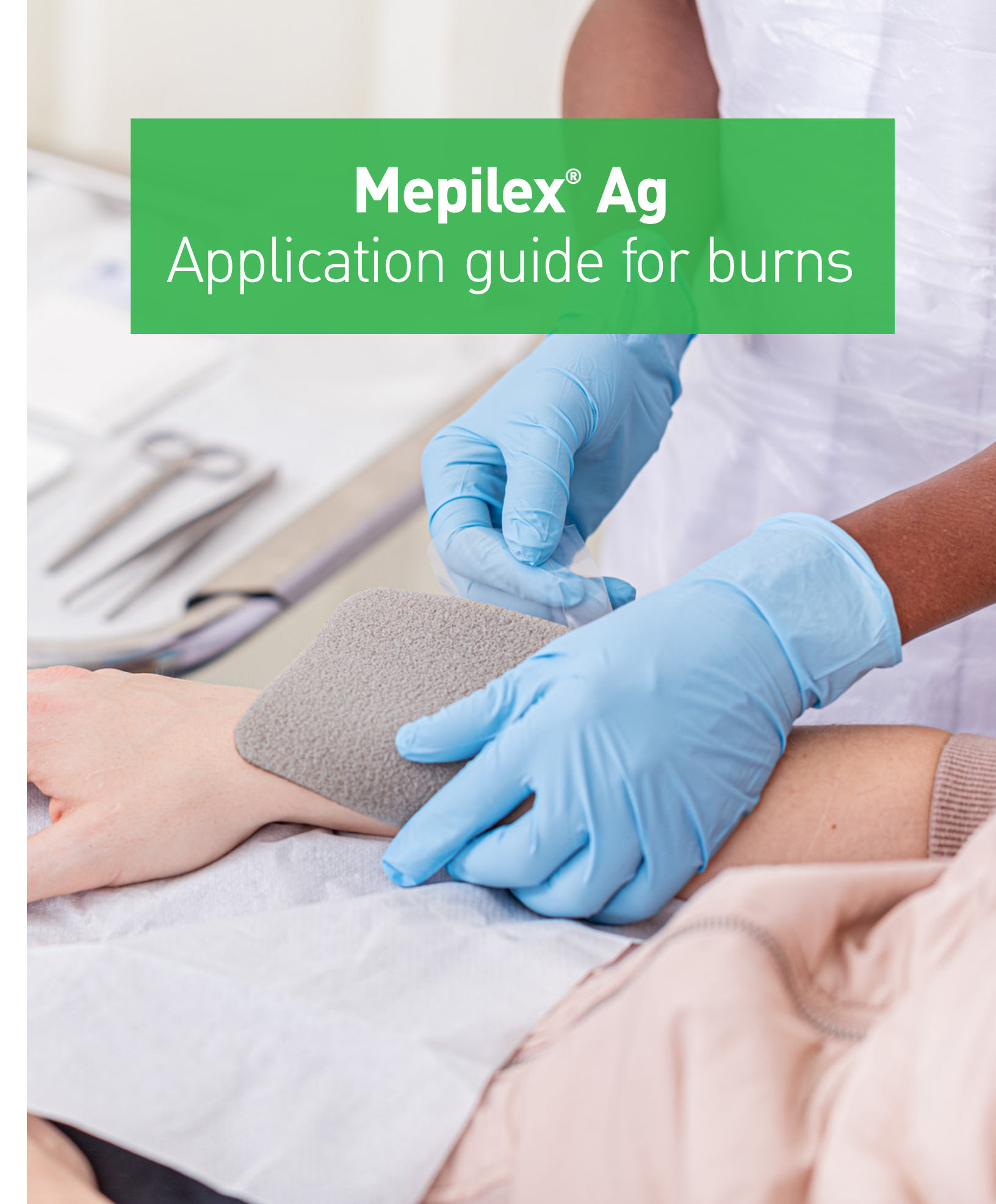

Mepilex® Ag
Application guide for burns

How to apply on the shoulder

For this application, you will need one Mepilex® Ag 10x20cm dressing.



1. Use Mepilex Ag 10x20cm.
2. Make two small, slightly misaligned cuts (about 1cm long) on each side of the dressing (see image).




3. When applying the dressing, create an overlap using the small cuts. This will enable greater freedom of movement for mobile body parts.

How to apply on the elbow and knee

For this application, you will need one Mepilex® Ag 10x20cm dressing.



1. Use Mepilex Ag 10x20cm.
2. Cut out two small triangles (about 1cm deep) on each side of the dressing. These cuts should be aligned.

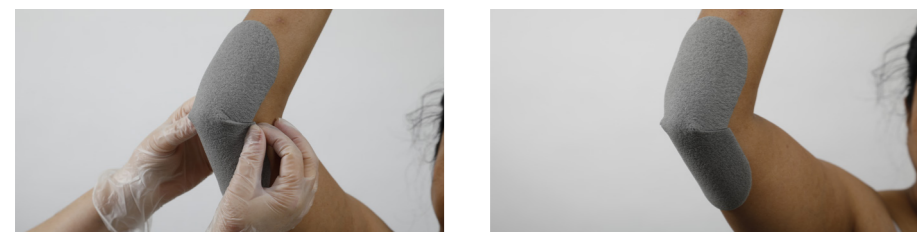

3. When applying the dressing, make sure the edges of the cuts are aligned to avoid leakage.

Tip:
All application techniques shown in this document are also applicable with **Mepilex® Transfer Ag**.

How to apply on the neck and chest

For this application, you will need three dressings in total: two Mepilex® Ag 10x20cm and one Mefix® 5cm width.

Step 1: Application on the neck



1. Use Mepilex Ag 10x20cm.
2. Cut the dressing lengthwise from end to end.
3. Cut two pieces of Mefix (about 5cm long).



4. Start by applying the first part closer to the face and then apply the second part aligned edge to edge with the first.
5. Secure the two dressing parts with one piece of Mefix (about 5cm long).

Step 2: Application on the chest



1. Use Mepilex Ag 10x20cm.
2. Cut the rounded end from the dressing to create a straight edge.
3. Apply the straight part of the dressing close to the dressing already placed on the neck. Make sure the edges are in contact with each other.



4. Secure the dressing parts with one piece of Mefix (about 5cm long).

How to apply on the face

For this application, you will need two Mepilex® Ag 10x20cm dressings.



1. Use 2 Mepilex Ag 10x20cm dressings.
2. Each dressing will cover one part of the face.
3. Identify where the eyebrow and the lower part of the eye are located.



4. Remove a part of the dressing to create space for the eye (about the length of index finger).
5. Identify where the mouth is located.



6. Cut out one small triangle (about 1cm long) to create space for the mouth.
7. Identify where the wing of the nose is located.
8. Cut out enough space in the dressing to accommodate the nose, where you identified the location of the wing of the nose in the previous step.



9. Now the dressing is cut to cover one side of the face; use this pattern to cut another dressing.
10. Use the part of the dressing you removed to cover the nose.