


Mepitel® | Mepitel® One

Product application guide

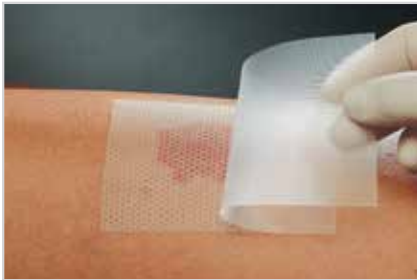

Mepitel + Mextra Superabsorbent

How to use Mepitel/Mepitel One:

1. Clean the wound area.
 2. Ensure peri-wound skin is dry.
- Note:** Moisten finger tips of gloves with normal saline for easy handling of Mepitel or Mepitel One
3. Remove smaller release film.
 4. Apply Mepitel or Mepitel One to the wound, allowing for at least 2 cm overlap of wound edges. Larger wounds may require more overlap.
 5. Remove upper release film (Mepitel only).
 6. Gels or topicals may be applied over Mepitel or Mepitel One as needed.
 7. Place absorptive dressing over Mepitel or Mepitel One dressing such as Mextra® Superabsorbent and a retention dressing, if needed, (i.e., Tubifast®, Tubigrip or Dermafit®).

Areas for use:



Injury to Finger



2nd Degree Burn



Epidermolysis Bullosa



Primary Wound Protection



Partial thickness burn



Fixation of meshed grafts



Fixation and protection of skin tear



IV Extravasation and infiltration

Areas of use

Skin Tears • Skin Abrasion • Surgical Incisions • Second Degree Burns • Blistering
Lacerations • Diabetic Ulcers • Venous and Arterial Ulcers • Partial and Full Thickness Grafts



Mepitel®

Safetac® technology layer on both sides

- Double sided Safetac technology for extra fixation of secondary dressings
- Good transparency for ongoing wound assessment
- Gentle Safetac technology for the most sensitive skin
- Conformable design ensures effective application and patient comfort
- May remain in place for up to 14 days for undisturbed healing
- May be used in conjunction with Negative Pressure Wound Therapy



Mepitel® One

Safetac technology layer on wound contact side only

- One sided Safetac technology offers easy application
- Full transparency allows for critical wound assessments without the need to disturb the dressing
- Stronger Safetac technology adhesion for extra security
- Highly conformable for excellent fit in difficult to dress locations
- May remain in place for up to 14 days for undisturbed healing

Mepitel ordering information‡

Product Code	Size	Pcs/box	Pcs/case	HCPCS
290799	3" x 4" (7.5 x 10 cm)	10	40	A6206
292005	8" x 12" (20 x 30 cm)	5	30	A6208

Mepitel One ordering information‡

Product Code	Size	Pcs/box	Pcs/case	HCPCS
289100	2" x 3" (5 x 7.5 cm)	10	70	A6206
289300	3" x 4" (7.5 x 10 cm)	10	40	A6206
289500	4" x 7" (10 x 18 cm)	10	70	A6207
289700	6.8" x 10" (17 x 25 cm)	5	40	A6208
289750	10.8" x 20" (27.5 x 50 cm)	2	14	A6208
289800	3.75" x 59" (9.5 x 150 cm)	1	9	-

Precautions

- The wound should be inspected for signs of infection according to clinical practice. Consult a healthcare professional for the appropriate medical treatment.
- Mepitel/Mepitel One used on burns treated with meshed grafts: avoid placing unnecessary pressure upon the dressing.
- Mepitel/Mepitel One used after facial resurfacing: avoid placing pressure upon the dressing. Lift and reposition the dressing at least every second day.
- When used on bleeding wounds or wounds with high viscosity exudate, Mepitel/Mepitel One should be covered with a moist absorbent dressing pad.
- When Mepitel/Mepitel One is used for the fixation of skin graft and protection of blisters, the dressing should not be changed before the fifth day post application.

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 1-800-843-8497 | www.molnlycke.us

The Mölnlycke, Mepitel and Safetac trademarks, names and logo types are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2018 Mölnlycke Health Care AB. All rights reserved. 1-800-843-8497. MHC-2018-36553

270264 02.18

